

# TECHNICAL DATA SHEET

### 58048 // PROTECTASEPT DUO WIPES

Type: Disinfection wipes, ready-to-use, neutral

Content: 48 pieces
Packaging Unit: 1,248 pieces





#### **Description**

Ready-to-use hygiene wipes for hand disinfection and for disinfection of alcohol-compatible surfaces, made of 100% biodegradable cellulose fibres with optimal active substance release and broad spectrum of activity, impregnated with disinfectant and very well tolerated by the skin, very practical and easy removal and application of the flowpack with resealable lid, "one-hand removal" possible through individual wipes, wipe size of 14 x 21 cm, flowpack with 48 wipes with neutral odour, efficacy according to EN 1500, EN 13727/13624, EN 14476, authorisation as biocide, distribution according to the Biocidal Products Directive only in Germany, use biocidal products carefully, always read label and product information before use, keep containers tightly closed, shelf life: 2 years, Standing time: 28 days, Biocide no.: N-96396

## **Preparation type**

Ready-to-use, pre-soaked hygiene wipes for disinfecting hands and disinfecting alcohol-compatible surfaces.

## **Application**

Clean heavily soiled hands and surfaces before disinfection. For surface disinfection, wet the surface completely and wipe with a second cloth if necessary. The surface must be kept moist during the entire exposure time. Observe the exposure times. For hand disinfection, wet the dry hands with the moist cloth and keep them moist for the entire exposure time; if necessary, use another cloth.

# Composition

In 100g impregnating liquid are contained: 0.8g didecyldimethylammonium chloride; 0.5g 2-phenoxyethanol

#### **Exposure times**

Efficacies of the soaking solution:

Hygienic hand disinfection EN 1500: 60 sec.

Surface disinfection: Bactericidal/levurocidal EN 13727/13624 (high load): 30 sec. Surface disinfection: Limited virucidal according to EN 14476 (low exposure): 60 sec.

#### Distribution

Distribution according to the Biocidal Products Directive only in Germany.