

58100 // PROTECTASEPT

Disinfectant wipes



For the universal disinfection of alcohol-tolerant surfaces

Scent: lemon

Reclosable dispenser box



58100 // PROTECTASEPT

Type: Disinfectant wipes, ready-to-use
Content: 100 wipes
Packaging unit: 12 piece(s)

NITRAS PROTECTASEPT, ready-to-use disinfectant wipes for the universal disinfection of alcohol-tolerant surfaces, polypropylene / viscose blend soaked with wipe disinfection, scent: lemon, IHO listed, sheet dimensions approx. 12 x 20 cm, reclosable dispenser box, efficiency tested according to EN 16615, dispenser box per 100 wipes, distribution in accordance with the Biocidal Products Ordinance only in Germany, Use biocidal products with care; always read the label and product information before use. Biocidal product number: N-75519 UFI: 1AFC-JN7C-H10P-K0H8



Industries

- Beauty Edition
- Black Edition
- Care Edition
- Dental Edition
- Food Edition
- Medical Edition

Advantages

- + For the universal disinfection of alcohol-tolerant surfaces
- + Scent: lemon
- + Reclosable dispenser box



Properties

Ready-to-use alcoholic wipes – selected lint-free nonwoven fabrics for excellent microbiological performance according to the EN16615 test procedure, with bonded fibers, extremely broad spectrum of activity for the highest demands, effective within 2 minutes under heavy use, practical and easy application and dispensing from the dispenser box, "one-handed dispensing" thanks to special sheet perforation with a large, pre-pulled core, convenient printing for maximum regulatory compliance with minimal documentation, long shelf life in the dispenser box (verified), evaporates quickly and leaves no residue, approved as a biocide and medical device (dual use).

Preparation type

Ready-to-use, pre-moistened wipes for the rapid disinfection of medical equipment and patient-contact and hygienically critical areas. The practical dispenser box allows for individual wipe removal and quick deployment.

Areas of application

According to the German Medical Devices Act (MPG): Alcoholic wipes for the prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for terminal disinfection of invasive medical devices.

According to the Biocidal Products Regulation: Alcoholic wipes for the rapid disinfection of alcohol-resistant surfaces.

Instructions for use

Wipe surfaces and objects until completely wet and allow the disinfectant to take effect. After the contact time has elapsed, wipe again with a disposable cloth if necessary. Suitable for use only on alcohol-compatible materials (not suitable for acrylic glass). The microbiological requirements for the quality of the rinsing water are at least drinking water quality.

Fibre composition

100g of impregnating liquid contains: 43.0g ethanol; 8.0g 1-propanol; 0.055g didecyltrimethylammonium chloride

Application times

Contact times for wipe systems:

Bacteria and *Candida albicans* according to EN 16615 with high contamination: Concentrated solution 2 min.

Based on the impregnation solution:

Bacteria according to EN 13727 and EN 13697 (high contamination): Concentrated solution 1 min.

Candida albicans according to EN 13624 and EN 13697 (high contamination): Concentrated solution 1 min.

TbB inactivation according to EN 14348 (*M. terrae*) with high contamination: Concentrated solution 1 min.

Enveloped viruses according to DVV/2012 in surface testing with high contamination: Concentrated solution 1 min.

Adenoviruses according to EN 14476 with high contamination: Concentrated solution 1 min.

Noroviruses according to EN 14476 with high contamination: Concentrated solution 1 min.

Rotaviruses according to EN 14476 with high contamination: Concentrated 1 min

Bactericidal (incl. MRSA), yeasticidal (*Candida albicans*), tuberculocidal (*M. terrae*), "limited virucidal PLUS" (effective against enveloped viruses such as HBV, HIV, HCV) and non-enveloped adenoviruses, noroviruses, rotaviruses

Safety instructions

H228 Flammable solid. Keep away from heat, sparks, open flames, and hot surfaces. No smoking. Wear protective gloves and eye protection. Store in a closed container.

Sales

Distribution in accordance with the Biocidal Products Regulation only in Germany

Additional information**Logistics data**

Size	GTIN (SPU)	GTIN (PU)	Length (m)	Width (m)	Height (m)	Weight (kg)	Quantity PU / pallet
One size	4059904065062	4059904065079	0,35 m	0,27 m	0,21 m	4,35 kg	72

Customs tariff number: 38089410

Country of origin: Germany