

58112 // PROTECTASEPT

Disinfectant wipes



For the universal disinfection of alcohol-tolerant surfaces

Scent: lemon

Refill bag



58112 // PROTECTASEPT

Type: Disinfectant wipes, ready-to-use
Content: 100 wipes
Packaging unit: 12 piece(s)

NITRAS PROTECTASEPT, ready-to-use disinfectant wipes for the universal disinfection of alcohol-tolerant surfaces, polypropylene / viscose blend soaked with wipe disinfection, scent: lemon, IHO listed, sheet dimensions approx. 12 x 20 cm, not reclosable refill bag, efficiency tested according to EN 16615, matching article 58100, polybag per 100 wipes, distribution in accordance with the Biocidal Products Ordinance only in Germany, Use biocidal products with care; always read the label and product information before use. Biocidal product number: N-75519 UFI: 1AFC-JN7C-H10P-K0H8



Industries

-  Beauty Edition
-  Black Edition
-  Care Edition
-  Dental Edition
-  Food Edition
-  Medical Edition

Advantages

- + For the universal disinfection of alcohol-tolerant surfaces
- + Scent: lemon
- + Refill bag



Properties

Ready-to-use alcoholic wipes - selected lint-free nonwoven fabrics for excellent microbiological performance according to the EN16615 test procedure with bonded fibers, extremely broad spectrum of activity for the highest demands within 2 minutes under heavy use, practical and easy application and dispensing from the dispenser box, "one-handed dispensing" thanks to special sheet perforation with a large, pre-pulled core, convenient printing for maximum regulatory compliance with minimal documentation, long shelf life in the dispenser box (tested), evaporates quickly and leaves no residue, approved as a biocide and medical device (dual use)

Preparation type

Ready-to-use, pre-moistened wipes for the rapid disinfection of medical equipment and patient-contact and hygienically critical areas. The practical dispenser box allows for individual wipe removal and quick deployment.

Areas of application

According to the German Medical Devices Act (MPG): Alcoholic wipes for the prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for terminal disinfection of invasive medical devices. According to the Biocidal Products Regulation: Alcoholic wipes for the rapid disinfection of alcohol-resistant surfaces.

Instructions for use

Wipe surfaces and objects until completely wet and allow the disinfectant to take effect. After the contact time has elapsed, wipe again with a disposable cloth if necessary. Suitable for use only on alcohol-compatible materials (not suitable for acrylic glass). The microbiological requirements for the quality of the rinsing water are at least drinking water quality.

Fibre composition

100g of impregnating liquid contains: 43.0g ethanol; 8.0g 1-propanol; 0.055g didecyltrimethylammonium chloride

Application times

Contact times for wipe systems: Bacteria and Candida albicans according to EN 16615 with high contamination: Concentrated solution 2 min. Based on the impregnation solution: Bacteria according to EN 13727 and EN 13697 (high contamination): Concentrated solution 1 min. Candida albicans according to EN 13624 and EN 13697 (high contamination): Concentrated solution 1 min. TbB inactivation according to EN 14348 (M. terrae) with high contamination: Concentrated solution 1 min. Enveloped viruses according to DVV/2012 in surface testing with high contamination: Concentrated solution 1 min. Adenoviruses according to EN 14476 with high contamination: Concentrated solution 1 min. Noroviruses according to EN 14476 with high contamination: Concentrated solution 1 min. Rotaviruses according to EN 14476 with high contamination: Concentrated 1 min Bactericidal (incl. MRSA), yeasticidal (Candida albicans), tuberculocidal (M. terrae), "limited virucidal PLUS" (effective against enveloped viruses such as HBV, HIV, HCV) and non-enveloped adenoviruses, noroviruses, rotaviruses

Safety instructions

H228 Flammable solid. Keep away from heat, sparks, open flames, and hot surfaces. No smoking. Wear protective gloves and eye protection. Store in a closed container.

Sales

Distribution in accordance with the Biocidal Products Directive only in Germany

Additional information

Logistics data

Size	GTIN (SPU)	GTIN (PU)	Length (m)	Width (m)	Height (m)	Weight (kg)	Quantity PU / pallet
One size	4059904065086	4059904065093	0,26 m	0,2 m	0,22 m	3,36 kg	80

Customs tariff number: 38089410

Country of origin: Germany