8100 // VINYL PF

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8100 // VINYL PF

Vinyl disposable gloves

PPE risk category I | Medical device class I

AQL < 1.5 (Performance Level 2, G1)

Expiry date

Not reusable

Marking medical device

MCD Medical Care Dental GmbH
Heinrich-Hertz-Str. 7
50181 Bedburg
Manufacturer

Phone: +49 2272 401 111
Mail: info@nitrax-medical.com
Web: nitrax-medical.com

EN

Manufacturer's instructions and information

Information brochure for personal protective equipment (PPE) according to Regulation (EU) 2016/425, annex II point 1.4. Please read this information brochure carefully before using the PPE. You are obligated to enclose this information brochure when passing on the PPE or to hand it over to the recipient of the PPE. For this purpose, this information brochure may be reproduced without restriction.

Vinyl disposable gloves	PPE risk category I Medical device class I
Size(s):	XS-XL
Colour code:	1400
Certification:	-

The CE marking certifies that the product complies with the essential health and safety requirements of Regulation (EU) 2016/425. The EU Declaration of Conformity can be viewed at doc.nitrax-medical.com.

This product is personal protective equipment in risk category I. It protects you against minor risks. These include only superficial mechanical injuries, contact with mildly aggressive/active cleaning agents or prolonged contact with water, contact with hot surfaces whose temperature does not exceed 50°C. Applications other than those mentioned above are expressly excluded. This product therefore does not offer protection against, among other things: mechanical risks, chemicals and microorganisms, cold, thermal risks (heat and/or fire), electric shocks, radiation, or working with high-pressure jets.

Storage / Use / Inspection: Store in a cool, dry place. Keep away from direct sunlight, UV rays and ozone sources. Do not store in a bent position or under heavy loads. Store and transport the product in its original packaging if possible. Factors such as light, moisture, temperature and natural material changes over a longer period of time can result in a change in the product properties. It is not possible to provide exact information on the storage time and service life of PPE, as both parameters depend, among other things, on the type of storage, temperature, humidity, degree of wear and intensity of use. Therefore, check this product for damage or material changes (e.g. brittle, cracked coatings/materials, holes, colour changes, etc.) after prolonged storage and before and after each use. Check this product before each use to ensure it is suitable for the intended activity and is the correct size. Unsuitable or defective products must be disposed of and must not be used under any circumstances. The size of the product may differ from the specifications, e.g. due to stretching.

Instructions for wearing the item: Ensure that your hands are clean and dry before putting on gloves. Insert your fingers into the respective gloves and pull the gloves loosely over your hands by the knitted cuffs or cuffs. Make sure they fit correctly. Gloves should fit snugly and tightly on the palm, fingers and between the fingers. Fingernails, jewellery and excessive stretching and pulling can damage the gloves. After use, gloves should be removed in such a way that the outside does not come into contact with clothing or skin, as these may be visibly or invisibly contaminated with harmful substances. Gloves should therefore be removed with the inside facing outwards. To do this, first loosen the fingertips of the glove from your fingers. The knitted cuff or cuff can then be turned inside out to remove the glove. To maintain the comfort of the glove, it should be cleaned after each use in accordance with the cleaning and maintenance instructions. Depending on requirements, this can and should be done while wearing the gloves.

Before starting work (after breaks and, if necessary, after washing hands), a suitable skin protection product can be used. During work (before breaks and before finishing work), a suitable skin cleanser can be used. After work (after the last hand wash), a suitable skin care product can be used.

Cleaning / Maintenance: This product is intended for single use only and must be disposed of regularly. Expiry date: see packaging.

Disposal: Dispose of this product with household waste. After intentional or unintentional contact with chemicals, this product may be contaminated with environmentally harmful or hazardous substances. In this case, disposal must be carried out in accordance with local regulations.

Special notes: PPE may cause allergic reactions in sensitive individuals. Special caution is advised in cases of known hypersensitivity.

Manufacturer

Year and month of manufacture

Recycling symbol (only for FR)

Read the manufacturer's instructions and information

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